

STOP THE PAIN AND START THE HEALING!



Carrington Laboratories, Inc.
2001 Walnut Hill Lane
Irving, Texas 75038
Phone: (800) 358-5205
International: (972) 518-1300
www.carringtonlabs.com
www.woundcare.com



SALICEPT™ Oral Patches

SaliCept™ Oral Patch is indicated for use in extraction sites and management of alveolar osteitis (dry socket). The SaliCept™ Oral Patch is also intended for the management of all types of oral wounds, injuries and ulcers of the oral mucosa. The SaliCept™ Oral Patch relieves pain by adhering to and protecting affected tissues from further irritation. It is safe if swallowed.

How supplied:
116003 12 patches/bx. 8 bxs./cs.

SALICEPT™ Oral Stomatitis Rinse

A freeze-dried powder, Acemannan Hydrogel™, for oral stomatitis that, when mixed with water, provides rapid relief from the pain of mouth ulcers by adhering to tissue and protecting against further irritation. It is soothing, non-stinging, non-irritating, and has a pleasant taste and smell. It is safe if swallowed.

How supplied:
101013 9.4 g 12/cs. (Makes approximately 7 fl. oz. when mixed with water.)
101101 2.35 g 36/cs. (Makes approximately 2 fl. oz. when mixed with water.)



SALICEPT™ Freeze-Dried Gel (FDG)

SaliCept™ FDG is an extract containing Acemannan Hydrogel™ with all water removed by freeze drying. It clings to mucous membranes, provides immediate pain relief and creates the ideal moist wound environment for healing. Its larger size makes it ideal for mouth ulcers - just tear off the needed size and shape. It is sterile, preservative-free, and tasteless. It is non-irritating, non-toxic and is safe if swallowed..

How supplied:
101090 4" dia. 15 ea./bag 4 bgs./cs.



 SaliCept™ products are for prescription use.

SaliCept® Oral Patch

Product Description:

The SaliCept® Oral Patch is a freeze-dried gel that contains Acemannan Hydrogel®, a product obtained from the inner gel of *Aloe vera* L. Other ingredients include hydroxethyl cellulose, polyvinylpyrrolidone, benzethonium chloride and simethicone. It is pliable, white to off-white, with a texture similar to that of finely woven cotton.

Protocol for Use:

Aphthous Stomatitis:

Clean and dry hands thoroughly. Remove patch being careful not to flatten patch between fingers. Gently press into place directly on ulcer being careful that it does not adhere to finger. (Patch will immediately adhere to moist tissue.) **Do not manipulate with tongue.** The patch will gradually rehydrate to a gel and continue to coat and protect tissues. (Care should be taken not to remove with food or drink.) Re-apply as needed.



SaliCept® Oral Stomatitis Rinse

Product Description:

SaliCept® is a mild, pleasant tasting, slightly viscous liquid when reconstituted. It is provided in a powdered form and mixed with water prior to use. SaliCept® relieves the pain of oral lesions by adhering to injured tissue of the oral mucosa and protecting it from further injury while providing a moist wound environment necessary for healing.

Protocol for Use:

1. Fill the bottle with water to the first arrow, then shake vigorously until all particles go into suspension. Fill the bottle with water to the second arrow and shake again to mix.
2. Swish or gargle using approximately one tablespoon of reconstituted SaliCept® liquid suspension in the mouth for at least one minute or as long as possible. Increased contact time enhances effectiveness. SaliCept® is safe if swallowed.
3. Swish or gargle four times a day, or more often, if needed. Again, SaliCept® is safe if swallowed.
4. Avoid eating or drinking for one hour after use, allowing SaliCept® to remain in contact with irritated tissue for as long as possible.



SaliCept® FDG

Product Description:

SaliCept® FDG is an extract containing Acemannan Hydrogel® with all water removed by freeze drying. Specifically designed for the management of mouth irritations caused by braces, partials, and ill-fitting dentures. Each pad is sterile, preservative-free, and tears easily into strips to place over irritated tissue.

Protocol for Use:

1. With clean, dry hands, remove SaliCept® Freeze-Dried Gel Pad and tear into strips.
2. Place SaliCept® strips on the affected areas within the mouth.
3. Allow packing to remain in place until it completely adheres to the affected areas and becomes a protective film. Safe if swallowed.
4. SaliCept® is a single use product. Discard any unused portion.

