

# Aphthous Stomatitis (Aphthous Ulcers, Canker Sores)

SaliCept™ Patch, containing Acemannan Hydrogel™, demonstrated a statistically significant reduction in discomfort scores within two minutes of application without the need for Lidocaine or Benzocaine and accompanying possible side effects.<sup>1</sup>



Multiple daily applications of SaliCept™ Patch, containing Acemannan Hydrogel™, to recurrent oral aphthae will coat and protect to allow rapid healing.<sup>1</sup>




**1** Peel the blister pack open to remove the patch.



**2** Clean and dry hands thoroughly. Remove patch being careful not to flatten patch between fingers.



**3** Locate ulcer. Make sure the area around the ulcer is clean and clear of food debris.

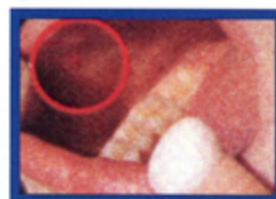

**4** Gently press into place directly on ulcer being careful that it does not adhere to finger. (Patch will immediately adhere to moist tissue.)



**5** The patch will stick to the ulcer once it is in place. Do not manipulate patch with tongue.



**6** The patch will gradually rehydrate to a gel and continue to coat and protect tissues. (Care should be taken not to remove with food or drink.) Re-apply as needed.